


## Features

- 100% Digital
- Volume Control
- Multi-Program Push Button (listening programs)
- Adaptive Noise Reduction
- Power On Delay
- Low Battery Indicator
- Program Indicator
- Replaceable Sound Tube
- Telephone Coil
- Adaptive Feedback Reduction
- Dynamic Speech Enhancement
- Hydro-coat moisture resistant barrier

## Identifying Your Device



The serial number is on the bottom of the case. The model is on the battery door. The first two digits of the serial number are the year of manufacture.

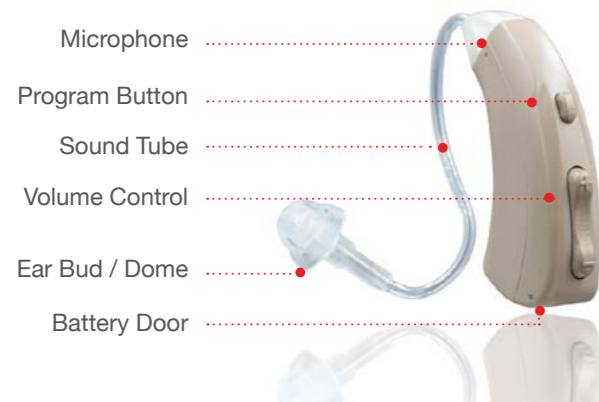

Call for customer support

**855-732-7636**

CE 0120

P E R S O N A<sup>®</sup>  
M E D I C A L

Persona Medical  
170 N. Cypress Way  
Casselberry, FL USA 32707  
Info@RxEars.com  
Tel: (407) 339-2422

ug\_rxears\_rx4\_rev-. 2\_19 ©



Welcome to a whole new world of hearing™

PRODUCT  
MANUAL

R X 4

**RxEARS**<sup>™</sup>  
HEAR...THERE...EVERYWHERE<sup>™</sup>



Welcome back to the world of sound!™

A belief that in order to succeed, you must have a certain passion and desire to give your every endeavor your very best. We forge this very commitment to excellence into every hearing instrument in our portfolio.



A rare combination of state of the art technology and custom craftsmanship brings you the best instruments available today. Currently, our products are distributed all over the world. We thank you for allowing us to assist with your hearing needs.


These instructions will help you attain maximum benefit from your hearing instrument(s).

## Cleaning & Changing Tubes


Use a soft cloth to wipe the unit after use to remove any particles or moisture. The cleaning tool is used to remove debris or wax from the sound tubing and earbud.




1. To remove tube, gently rotate the sound tubing holding the instrument firmly.

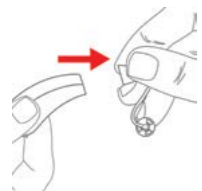

2. Pull sound tubing from the hearing instrument.

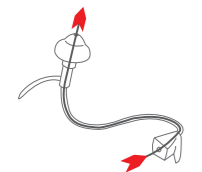

3. To clean, insert the cleaning tool through the nose cone end. Thread all the way through the tubing and earbud. Use brush supplied to brush off any ear wax.

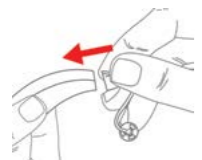

4. To reassemble or change sound tubing simply line up and push tubing until it snaps into place.

## Powering Your Device

### Batteries - Uses a 312 battery

**TO TURN ON** - With your finger, gently open the door. Insert battery and close the door. Your device is equipped with a power on delay, you will have five seconds to seat the device comfortably in your ear before it turns on.

When placing the battery in your device, position the battery with the positive (+) side (or totally flat side) up and insert it into the battery compartment section of the battery door. If the battery is inserted incorrectly or upside down, the door will not close. Do not force the door to close.

**TO TURN OFF** - To remove the device simply grasp the tubing close to the ear tip (dome / ZipTip) and remove from ear canal. Then remove instrument from the top of the ear. To turn your device off, simply open the battery door completely. Store in a safe dry place.

**CAUTION: Batteries are dangerous if swallowed! Keep batteries out of the reach of children. Never put batteries in your mouth for any reason. They are easily swallowed. In the event a battery has been swallowed, seek immediate medical attention. For more information call the National Battery Hotline at 202-625-3333.**

**NOTE: Some irritation may occur due to becoming accustomed to wearing a new hearing instrument. If swelling or redness occurs, discontinue use and visit your Doctor.**

## Inserting Your Device

Before inserting, please clean your ear and make sure the battery door is closed properly.

- Hold the tubing with your thumb and index finger close to the ear dome (Fig. A).
- Place the instrument over the top of the ear. (Fig. A)
- Gently insert the ear dome into your ear canal. (Fig. B)
- Move the retention lock into position by lifting and placing it into the bowl of your ear. (Fig. C)



Fig. A



Fig. B



Fig. C

### Earbuds / Tips

- D. Tube with earbud. Use this tip if ZipTip does not fit or if you feel too "plugged up."
- E. Tube with ZipTip - Keep the small vent hole (F) to the bottom of the ear farther from the device



## Volume Adjustment

The rocker switch controls your volume. Place your index finger on the rocker switch. (Fig. 1). Briefly push/release the top half of the switch to increase volume (Fig. 2) and bottom half of switch to decrease the volume (Fig. 3). **Holding the button down continuously will not change the volume; it will change the program.**

Fig. 1



Fig. 2


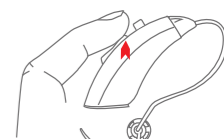


Fig. 3



Fig. 4

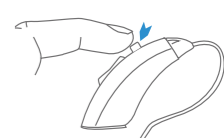

### Program Adjustment

After closing the battery door, the device turns on in the program last used. By pressing the small top button (Fig. 4), the programs will cycle up in sequence 1,2,3,4,1,2,3,4 etc...

- Program 1: Daily Listening/Conversation. Quiet environments.
- Program 2: A little less bass.
- Program 3: Crowded Rooms. Minimum bass, maximum treble.
- Program 4: Telecoil. Used for telephones and loop systems.