


## Identifying your hearing instrument

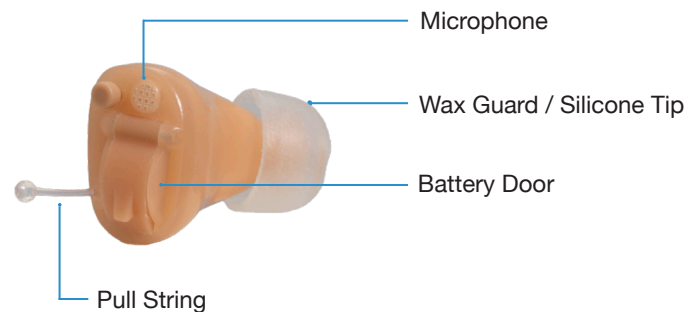

Every instrument has a unique identifier. The first two numbers indicate the year of manufacture, the next six numbers indicate the serial number. The model name is on the opposite side of the case.

Company/Brand  
YR 123456

**Wax Protection:** It is very important to keep the wax guard clean. Gently pull off the silicone tip, and use a soft brush or wax removal tool to remove any accumulated debris. The wax guard will occasionally need replacing. The Hear Clear wax guard is used on your hearing instrument.

Call for customer support

**855-732-7636**

CE 0120

P E R S O N A<sup>®</sup>  
M E D I C A L

Persona Medical  
170 N. Cypress Way  
Casselberry, FL USA 32707  
Info@RxEars.com  
Tel: (407) 339-2422

ug\_rxi\_rev-A 2\_19 ©



Welcome to a whole new world of hearing™

Welcome back to the world of sound!™

A belief that in order to succeed, you must have a certain passion and desire to give your every endeavor your very best. We forge this very commitment to excellence into every hearing instrument in our portfolio.



PRODUCT  
MANUAL

R X I

**RxEARS**™  
HEAR...THERE...EVERYWHERE™



A rare combination of state of the art technology and custom craftsmanship brings you the best instruments available today. Currently, our products are distributed all over the world. We thank you for allowing us to assist with your hearing needs.

These instructions will help you attain maximum benefit from your hearing instrument(s).

## Powering your hearing instrument

### Batteries: This instrument uses a 10A battery.

**ON** - With your finger, gently open the door. Insert battery and close the door. You will have several seconds to seat the instrument comfortably in your ear before it turns on.



When placing the battery in your instrument, be sure to position the positive (+) symbol on the side of the battery to line up with the positive (+) symbol shown on the battery door.


**OFF** - To turn your hearing instrument off, simply open the battery door wide enough so that the internal contacts are no longer touching the battery. Store in a safe dry place.

**CAUTION:** Batteries are dangerous if swallowed! Keep batteries out of the reach of children. Never put batteries in your mouth for any reason as they are easily swallowed. In the event a battery has been swallowed, seek immediate medical attention. For more information call the National Battery Hotline at 202-625-3333.

## Inserting / Removing

### Inserting your hearing instrument

Before insertion, make sure the battery door is closed properly. To insert your instrument, hold it with your thumb and index finger. Slowly guide the instrument into your ear canal until you feel resistance. Gently push the instrument into your ear until it sits comfortably in your canal.

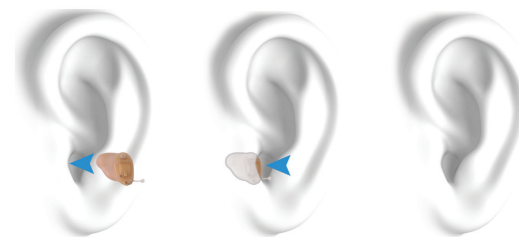

To ease insertion, you may wish to pull up slightly on the top of your ear with your other hand as you push the hearing instrument into your ear.


### Removing your hearing instrument

To remove, gently pull on the instrument by the pull string with your thumb and index finger, do not pull by battery door. Don't forget to turn the instrument off, by opening the battery door after you remove it from your ear.

## Physical Adjustments

### Tips

Three differently sized tips are enclosed. These can be used to reduce any whistling-feedback or improve comfort. The tips are pushed onto the tip of the device. Please ensure you have pushed the tip onto the instrument as far as possible before inserting into your ear.

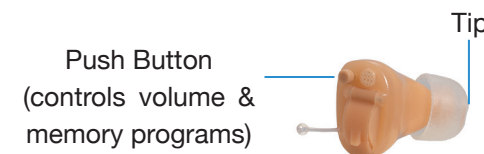

### Memory / Program Adjustment

Your unit has a Multi-Memory Push Button to select up to four different listening situations. Push and hold the button down for two seconds, and the program will cycle to the next memory. The number of beeps you hear corresponds to the memory/program set. The programs will cycle in sequence 1,2,3,4,1,2,3,4...etc.

Program 1: Daily Listening/Conversation. Max bass.  
Program 2: A little less bass.  
Program 3: Less bass, more treble.  
Program 4: Min bass, max treble for noisy environments.

## Volume Control

### Volume

Each time your device is turned on, it will automatically set to the previously used volume setting.

To adjust the volume, tap the push button down and release quickly. You will hear a beep tone. Listen to the volume of conversation and if you need it louder, repeat the process. Each short press of the push button will increase the volume slightly. Once the maximum volume is attained, you will hear three beeps. The next sequential press of the button will cycle back to minimum volume.

### Volume Control Cycle

